
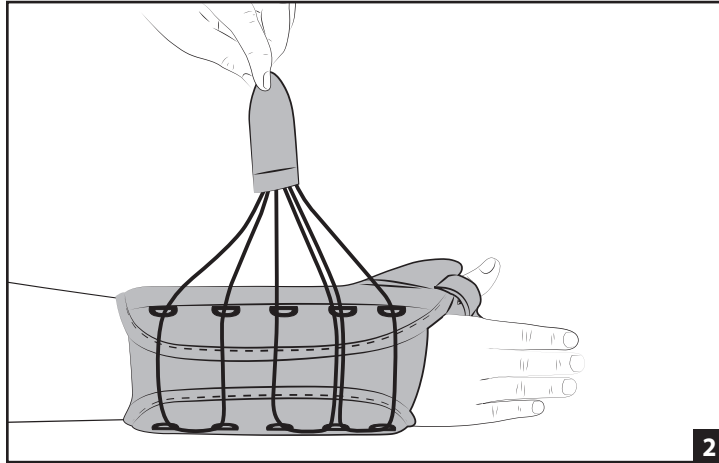
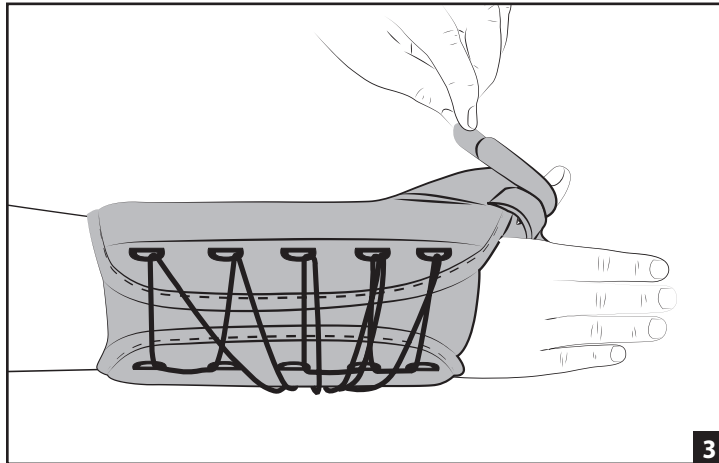
1

2

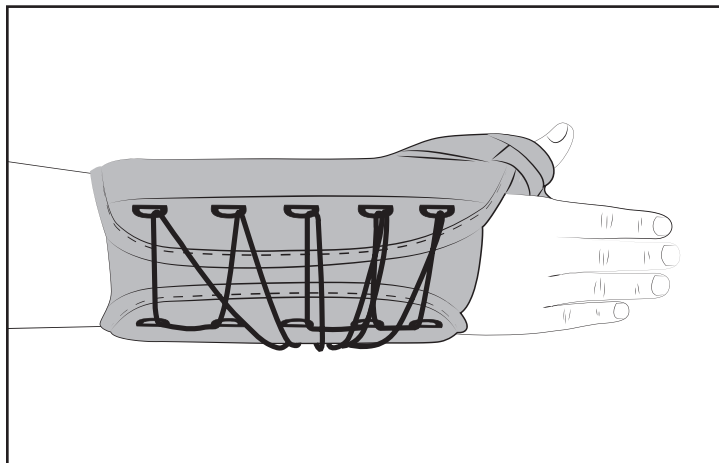3

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS: Treatment of deQuervains syndrome; Scaphoid injuries; Gamekeeper's thumb; and Wrist and thumb sprains and strains.

PERFORMANCE CHARACTERISTICS: Soft-good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS: None

WARNINGS AND PRECAUTIONS:

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

- 1) Release single pull strap and thumb straps. Position hand in brace with cotton stockinet liner on top of hand. Thumb should rest on the abduction stay. Make sure the Palmer stay is positioned comfortably below the palmer crease and aligned with forearm.
- 2) Wrap single pull strap around wrist area and secure on brace. Strap should be snug but not constrict.
- 3) Thumb straps should wrap in opposing directions and secure on brace. Straps should be snug but not constrict.

CLEANING INSTRUCTIONS:

Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

WARRANTY: DJO, LLC will repair or replace all or part of unit and its accessories for material and workmanship defect for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY.

NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2021 DJO, LLC