# AIRCAST® AIRSELECT™ STANDARD

|                  |    |
|------------------|----|
| ENGLISH .....    | 2  |
| ESPAÑOL .....    | 4  |
| DEUTSCH.....     | 6  |
| ITALIANO.....    | 8  |
| FRANÇAIS .....   | 10 |
| NEDERLANDS.....  | 12 |
| SLOVENČINA ..... | 14 |
| DANSK .....      | 16 |
| SVENSKA.....     | 18 |
| SUOMI .....      | 20 |
| ČEŠTINA .....    | 22 |
| PORTUGUÊS.....   | 24 |
| NORSK.....       | 26 |
| 简体中文 .....       | 28 |
| 日本語 .....        | 30 |

AIRCAST®







DJO, LLC  
A DJO Global Company  
1430 Decision Street  
Vista, CA 92081-8553 • USA

T 760.727.1280 • F 760.734.5608  
DJOglobal.com

©2014 DJO, LLC

01E961\_B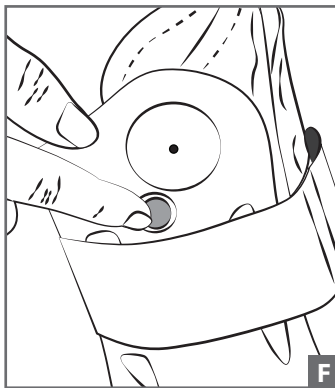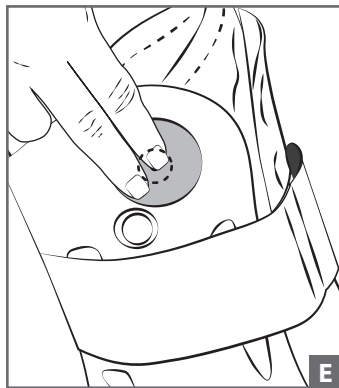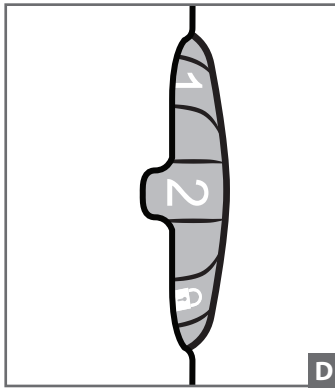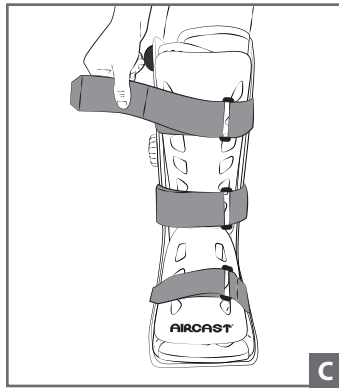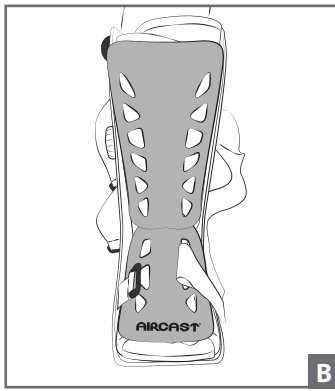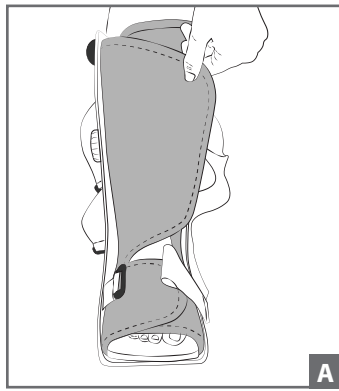

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USE / INDICATIONS:

The STANDARD walking brace provides compressed air support for stable fracture of foot and/or ankle, severe ankle sprains, and post-operative use. Effective in fracture healing and the management of edema (swelling).

### WARNINGS AND PRECAUTIONS:

Do not over-inflate aircells. Improper aircell inflation may cause significant skin irritation, especially in people with diabetic neuropathy (a diabetic nerve condition). Check your skin frequently for "hot spots" and skin irritation. If you see any skin changes or feelings of discomfort, reduce the aircell compression. Do not use this device if you cannot feel sensations whether due to post-op anesthesia, nerve, skin or other conditions.

Use caution when walking on slippery or wet surfaces to avoid injury.

### APPLICATION INFORMATION:

*A cotton or cast sock will enhance comfort – NOT INCLUDED*

#### 1) PREPARE BRACE –

- Unfasten straps, remove front panel, and open liner.

#### 2) APPLY BRACE –

- While seated, place foot in brace positioning the heel against the back of the brace. Wrap liner around foot and then leg (Fig. A).
- *Replace front panel:* Position front panel so the Aircast logo is over the toes and the upper part of the panel is secured to the liner (Fig. B).
- Secure straps from bottom to top (Fig. C). Tighten until snug and comfortable.

#### 3) ADJUST AIRCELL COMPRESSION –

*The STANDARD walking brace aircells are initially deflated*

- Begin inflation on injured side first. Turn the SELECTOR found on the right-side of the brace to the appropriate number (Fig. D). Number 1 inflates the aircell on the right-side and Number 2 inflates the aircell on the left-side. The "lock" position should be used once both aircells are inflated. This prevents accidental deflation.
- To inflate aircell, place finger over small hole on top of **large** inflation pump found on the top right side of brace (Fig E). Pump until aircell feels snug.
- To inflate other aircell, rotate SELECTOR to the appropriate number and then press and release large inflation pump. Pump until aircell feels snug. When both aircells are inflated, rotate SELECTOR to "lock" position.
- In the event of over-inflation, deflate aircells.

#### 4) DEFLATE AIRCELLS –

- To deflate aircells, turn the SELECTOR found on the right-side of the brace to the proper aircell number (Fig. D).
- Press and hold finger on **small** release button found on the top right-side of brace (Fig. F).
- Deflate until pressure is comfortable.

NOTE: Deflate aircells **before removing brace** for proper fit when reapplying.

NOTE: Higher pressure provides more support when walking, lower pressure is more comfortable when sitting or reclining. Pressure can also be adjusted by tightening or loosening straps.

NOTE: The shell can be trimmed with a cast saw or cast scissors to accommodate larger calves.

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### **CLEANING INSTRUCTIONS:**

THE FOAM LINER MAY BE HAND-WASHED IN 86°F/30°C WATER WITH MILD DETERGENT AND AIR-DRIED. DO NOT PUT IN DRYER OR USE OTHER HEAT SOURCE TO DRY.

### **HIGH ALTITUDE:**

AT HIGH ALTITUDES THE AIRCELLS WILL EXPAND BEYOND THEIR OPTIMAL LEVEL. ADJUST AIRCELL PRESSURE BY TIGHTENING OR LOOSENING THE STRAPS.

### **WARRANTY:**

DJO Global, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

***NOT MADE WITH NATURAL RUBBER LATEX***

**INTENDED FOR USE ON A SINGLE PATIENT**

### **MATERIAL COMPONENTS**

Shell: POLYPROPYLENE

Front Panel: HIGH DENSITY POLYETHYLENE

Straps: NYLON

Inflation Pumps: THERMOPLASTIC POLYURETHANE

Liner: NYLON/POLYURETHANE/POLYESTER (OUTER/FOAM/INNER MESH)

Binding: POLYESTER

Outer Sole / Insole: EVA/RUBBER

Selector dial: NYLON

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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