# SAFB-OT

## INSTRUCTIONS

### SHORT ARM FRACTURE BRACE OPEN THUMB

antimicrobial treated

no. 30094 Rev E  
USA and International Patents Pending

Exos LLC  
1122 Red Fox Road  
Arden Hills, MN 55122

Medical Device & QA Services Ltd  
75, Stockport Road, Timperley,  
Cheshire, WA15 7SN, United Kingdom  
Tel +44 161 870 6751 Fax +44 161 903 9787  
Email: info@mdqservices.com  
www.mdqservices.com

### SHORT ARM FRACTURE BRACE OPEN THUMB (SAFB-OT)

ORTESIS CORTA PARA FRACTURAS DEL BRAZO CON PULGAR ABIERTO  
EXOS™ HAND- / UNTERARMORTHESE MIT BOA®, OFFEN  
TUTORE PER FRATTURA DELL'AVAMBRACCIO CON POLLICE APERTO  
ATTELLE COURTE POUR FRACTURE DE BRAS À POUCE OUVERT  
KORTE ARM FRACTUUR BRACE MET OPEN DUIM  
KRÁTKA ORTEZA RUKY NA ZLOMENINY S OTVOROM NA PALEC  
SAFB-OT-SKINNEN  
SHORT ARM FRACTURE BRACE OPEN THUMB  
SHORT ARM FRACTURE BRACE OPEN THUMB  
KRÁTKA ORTEZA PRO FRAKTURY PÁČI S VOLNÝM PALCEM  
TALA CURTA PARA FRACTURA DO BRAÇO COM POLEGAR ABERTO  
DEN KORTE ARMFRAKTURSKINNEN MED ÅPEN TOMMEL  
振露拇指短臂骨折器具  
ショート腕骨折用ブレース（自在に親指を動かせるタイプ）

no. 30094 Rev E  
USA and International Patents Pending

Exos LLC  
1122 Red Fox Road  
Arden Hills, MN 55122

Medical Device & QA Services Ltd  
75, Stockport Road, Timperley,  
Cheshire, WA15 7SN, United Kingdom  
Tel +44 161 870 6751 Fax +44 161 903 9787  
Email: info@mdqservices.com  
www.mdqservices.com

Medical Device & QA Services Ltd  
75, Stockport Road, Timperley,  
Cheshire, WA15 7SN, United Kingdom  
Tel +44 161 870 6751 Fax +44 161 903 9787  
Email: info@mdqservices.com  
www.mdqservices.com

1. Loosen or remove the brace.
2. Wash inside and outside of brace and skin with antibacterial soap and water (86°F / 30°C).
3. Rinse thoroughly with water.
4. Use a hair dryer set on high volume and cool setting to thoroughly dry the interior of the brace and skin.
5. Retighten the brace.
6. For fitting tips on each Exos™ device: Visit the Exos™ Provider Center at: <http://exosmedical.com/provider/SAFB-OT> or Scan the QR image with your smartphone
7. Material Used: EVA foam, spandex, nylon, terpolymer, polyester, steel, plastic, nylon 6/6 resin, Antimicrobial treatment

**LIMITED PRODUCT WARRANTY**  
Exos™ covers defects in workmanship and materials up to 8 weeks from date this product is dispensed. Damage caused by normal wear and tear, abuse, or alterations is not covered.

## ENGLISH

Before using the device, please read the entire instruction manual. Correct application and care is vital to the proper functioning of the device.

### INTENDED USE / INDICATIONS

The Short Arm Fracture Brace Open Thumb (SAFB-OT) provides stabilization to the distal radius, distal ulna, and wrist joint.

### CONTRAINDICATIONS

Patients with known allergies to foams, adhesives, or plastic products.

### ⚠️ WARNINGS and PRECAUTIONS

- ▶ Brace should be fitted by a qualified healthcare professional.
- ▶ Do not heat brace on the body.
- ▶ Check to make sure brace is not too hot prior to application.
- ▶ Brace should be appropriately sized to the patient to ensure proper support and function.
- ▶ Care should be taken with patients with delicate or compromised skin, especially the elderly, not to apply the brace too tightly, which MAY cause pressure and shear to the fragile tissue. PATIENTS SHOULD BE INSTRUCTED ON APPROPRIATE TENSIONING DURING ONGOING USE. An under sleeve or STOCKINETTE may be used for added protection.
- ▶ Do not heat Exos™ products to greater than 212°F (100°C).
- ▶ FOR SINGLE PATIENT USE ONLY.

### OPERATING BOA® LACING AND RELEASE TAB SYSTEM

#### To Loosen Boa®:

- ▶ Lift Boa® Knob. (Figure 1)
- ▶ Pull Tab. (Figure 2)

#### Release Tab:

- ▶ To allow the brace to open more fully, pull open the hook and loop release tab and free lace. (Figure 3)
- ▶ Prior to tightening brace, place the lace back into the hook and loop release tab and secure hook to loop. (Figure 4)

#### To Close and Tighten Boa®:

- ▶ Push Boa® Knob down. (Figure 5)
- ▶ Turn Boa® Knob clockwise. (Figure 5)
- ▶ Boa® Knob must be in the down/locked position when wearing brace.

### HEATING AND APPLICATION

- ▶ Open and unfold brace.
- ▶ Proper heating temperature is 200°F (93°C).
- ▶ Heat brace in Exos™ recommended oven approximately 5 minutes or until brace is pliable. Brace should not be too hot to handle comfortably.
- ▶ Ensure that skin is completely dry before application.
- ▶ Fit brace on patient.
- ▶ Remove slack in lace by tightening Boa® Knob.
- ▶ **Caution: Do not tighten brace fully until brace has returned to room temperature otherwise indentations in plastic may occur.**
- ▶ Mold brace into desired position. (Figure 6)
- ▶ Accessory strap may be secured in the desired position. (Figure 7)
- ▶ Once brace has hardened, the Boa® Knob may be tightened to desired tension.
- ▶ **Warning: The brace should not be worn too tightly. This may produce excessive mechanical force or shear forces to the skin.**

### CLEANING INSTRUCTIONS

1. Loosen or remove the brace.
2. Wash inside and outside of brace and skin with antibacterial soap and water (86°F / 30°C).
3. Rinse thoroughly with water.
4. Use a hair dryer set on high volume and cool setting to thoroughly dry the interior of the brace and skin.
5. Retighten the brace.

#### For fitting tips on each Exos™ device:

Visit the Exos™ Provider Center at: <http://exosmedical.com/provider/SAFB-OT> or Scan the QR image with your smartphone

### MATERIAL USED

EVA foam, spandex, nylon, terpolymer, polyester, steel, plastic, nylon 6/6 resin, Antimicrobial treatment

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