**ENGLISH**


**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USER PROFILE:** THE USER SHOULD BE ABLE TO READ AND UNDERSTAND THE DIRECTIONS, WARNINGS AND CAUTIONS.


**INDICATIONS:** ACL, PCL, ACL/PCL Deficiencies - ACL, PCL, ACL/PCL reconstructions - Collateral ligament deficiencies - Collateral ligament reconstructions with PCL involvement - MCL and/or LCL sprains - Hyperextension

This product was designed to complement the variety of medical treatments commonly used for the knee. The frequency and duration of use should be determined by your prescribing Healthcare professional.

- CONTRAINDICATIONS:** None
- APPLICATION INFORMATION:**
- A: While sitting on the edge of a chair, step through the brace. Flexing the knee to 45°, position the brace as shown in figure 1, centering the hinge about 1" (2.54 cm) above the top of the kneecap and slightly past midline of the leg. Make sure the brace is not twisted on the leg. B: Push the bottom of the brace up until the bottom of the thigh strap (#2).
  - C: Keeping the knee bent at 45 degrees, push both condyles (hinges) toward the back of the leg, as shown in figure 2.
  - D: Maintaining the position of the hinges for ACL version secure lower thigh strap (#3); for PCL version secure upper calf strap (#5); for PCL version secure lower thigh strap (#3) then upper calf strap (#4) on the back of the leg.
  - E: For ACL version, draw the hinges forward slightly before fastening the front tibial (shin) strap (#4). For PCL version draw the hinges forward slightly before fastening the front thigh strap (#4). For CI version draw the hinges forward slightly before fastening the front tibial (shin) (#5) and the front thigh strap (#6). Do not pull the hinges beyond midline of the leg.
  - F: For ACL version use upper calf strap (#5). Be careful not to pull this strap too tight and risk circulation problems.
  - G: When all strapping is complete, the hinge should center just above the top of the kneecap and slightly behind midline. To ensure that no slippage occurs during activity, it is recommended to use a few knee bends or walk for a few minutes, then re-tighten the straps following steps B-F or B-E for CI version.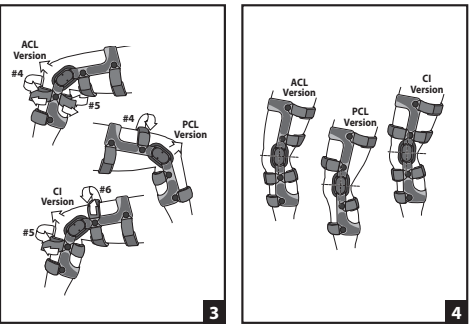

**INSTRUCTIONS FOR HINGE ADJUSTMENT**  
Your brace comes with 10° extension stops installed. To change the installed stops, first completely remove the screw as shown for Armor by DonJoy & Legend figure 5, for 4TTUDE, 4TTUDE Advanced & Renegade, figure 6. Then remove the stops, insert the new stops, align with the holes in the hinge plate, then reinsert the screw. Using a Phillips head screwdriver, ensure that the screws are seated in the recess and completely tightened. Both medial and lateral stops must be identical in degree.

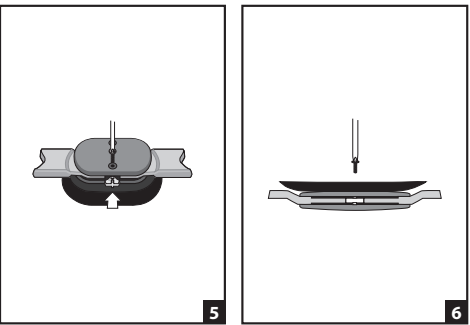Lateral side shield hinge cover (standard on LEGEND). In order to remove the extension stop, it is necessary to flex the hinge and slide the protective shield off cover before removing the stop (figure 7).

Extension Stops: 0°, 10°, 20°, 30°, and 40°. Flexion Stops: 0°, 45°, 60°, 75°, and 90° (no flexion stops are installed upon shipment).

**USE AND CARE**

- If during activity, re-tightening is necessary, re-tighten all straps following steps in the application instructions.
- All straps and pads may be trimmed to fit individual variations in limb circumference.
- The brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water and then air dry.
- Hand wash your liners and condyle pads in cold water with mild detergent. AIR DRY ONLY. DO NOT HEAT DRY LINERS OR PADS.
- The hinge mechanism requires little maintenance in normal use. However, you may apply a dry lubricant such as a Teflon™ spray to the internal mechanism of the hinge. Dry lubrication is recommended following use in water.
- Periodically inspect hinge screws for tightness and retighten as needed.

**Always consult with your physician or therapist before making changes to the brace.**


**CLEANING INSTRUCTIONS:** Hand wash in water (86°F/30°C) using mild soap. Air dry.

**MATERIAL CONTENTS:**  
Frame - Aluminium  
Hinge - Aluminium & Steel  
Strap Tabs - Aluminium & Nylon  
Soft Goods - Nylon & EVA

**WARNING:** THIS PRODUCT IS TO BE USED UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. THIS DEVICE IS NOT INTENDED FOR UNSUPERVISED PUBLIC USE. IF YOU EXPERIENCE ANY PAIN, SWELLING, SENSATION CHANGES, OR ANY UNUSUAL REACTIONS WHILE USING THIS PRODUCT, CONSULT YOUR MEDICAL PROFESSIONAL IMMEDIATELY.

**WARRANTY:** DJO, LLC will repair or replace all or part of the product and its accessories for material or workmanship defects for a period of six months for softgoods and one year on brace frame/hinges from the date of sale.

**For single patient use only. Not Made with Natural Rubber Latex**

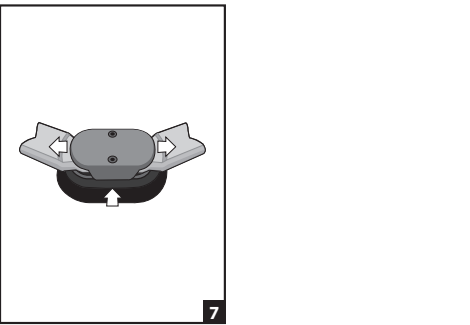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