AIRCAST® AIRSELECT™ SHORT

ENGLISH	2
ESPAÑOL	4
DEUTSCH.....	6
ITALIANO.....	8
FRANÇAIS	10
NEDERLANDS.....	12
SLOVENČINA	14
DANSK	16
SVENSKA.....	18
SUOMI	20
ČEŠTINA	22
PORTUGUÊS.....	24
NORSK.....	26
简体中文	28
日本語	30

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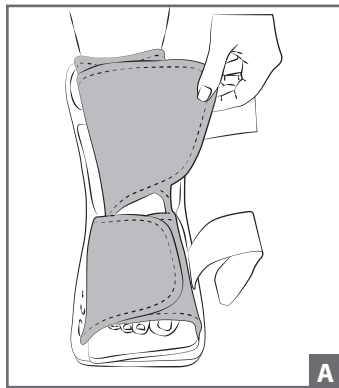
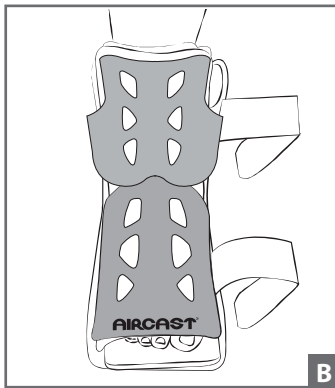
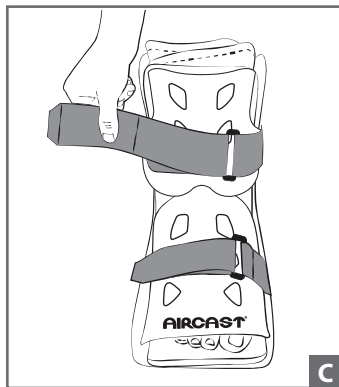
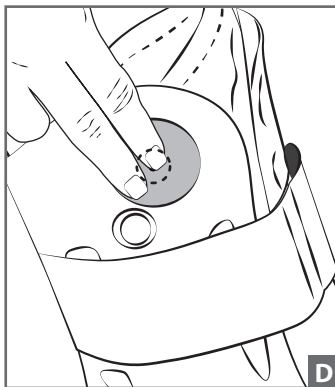
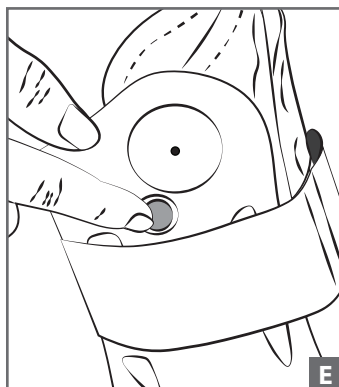
E

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE / INDICATIONS: The Aircast AirSelect Short walker is designed to provide support and immobilization to the ankle. It may be suitable for the treatment of stable fractures of the foot, lateral ankle sprains and ligamentous injuries such as Plantar Fasciitis. Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS: Do not use if you are allergic to any of the materials contained within this product. This device is contraindicated for unstable fractures of the lower leg, ankle and foot.

WARNINGS AND PRECAUTIONS:

- This device is to be used under the supervision of a healthcare professional. The determination of when to apply the brace and the frequency and duration of use should be strictly at the discretion of the treating physician. Always consult with your physician before making changes to the brace.
- Do not use this device on patient's incapable of communicating physical discomfort.
- Do not over-inflate aircells. Improper aircell inflation may cause significant skin irritation, especially in people with diabetic neuropathy. Reduce aircell pressure with any visual skin changes or reported discomfort.
- Patient's without sensation (i.e. post-op anesthesia, neuropathies, etc.) should be monitored frequently for skin damage, skin irritation or wound management.
- Use caution when walking on slippery or wet surfaces to avoid injury.
- Do not over tighten straps. This may result in reduction of blood flow or sensation.
- Do not use over open wounds.
- Do not modify the device or use the device other than intended.
- At high altitudes the aircells will expand beyond their optimal level. Adjust aircell support by tightening or loosening straps.
- Do not use this device if it is damaged and/or the packaging has been opened.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, consult your medical professional immediately.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

A cotton or cast sock will enhance comfort – NOT INCLUDED

- 1) PREPARE BRACE –**
 - Unfasten straps, remove front panel, and open liner.
- 2) APPLY BRACE –**
 - While seated, place foot in brace. Position heel in back of brace. Wrap liner around foot and then ankle (Fig. A).
 - *Replace front panel:* Position front panel so the Aircast logo is over the toes and the foot panel is situated inside the brace. The upper part of the panel should be secured outside the brace (Fig. B).
 - Secure straps from bottom to top (Fig. C). Tighten until snug and comfortable.
- 3) ADJUST AIRCELL SUPPORT –**

The SHORT walking brace aircells are initially deflated

 - Begin inflation on injured side first.
 - To inflate aircell, place finger over small hole on top of **large** inflation pump found on either side of brace (Fig D). Pump until aircell feels snug.
 - Repeat on other side. Pump until aircell feels snug.
 - In the event of over-inflation, deflate aircells.
- 4) DEFLATE AIRCELLS –**
 - To deflate aircells, press and hold finger on **small** release button (Fig. E).
 - Deflate until pressure is comfortable.

NOTE: Deflate aircells **before removing brace** for proper fit when reapplying.

NOTE: Higher pressure provides more support when walking, lower pressure is more comfortable when sitting or reclining. Pressure can also be adjusted by tightening or loosening straps.

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CLEANING INSTRUCTIONS:

THE FOAM LINER MAY BE HAND-WASHED IN 86°F/30°C WATER WITH MILD DETERGENT AND AIR-DRIED. DO NOT PUT IN DRYER OR USE OTHER HEAT SOURCE TO DRY.

WARRANTY:

DJO Global, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOT MADE WITH NATURAL RUBBER LATEX.

INTENDED FOR USE ON A SINGLE PATIENT

RX ONLY

MATERIAL COMPONENTS

Shell: POLYPROPYLENE

Front Panel: HIGH DENSITY POLYETHYLENE

Straps: NYLON

Inflation Pumps: THERMOPLASTIC POLYURETHANE

Liner: NYLON/POLYURETHANE/POLYESTER (OUTER/FOAM/INNER MESH)

Binding: POLYESTER

Outer Sole / Insole: EVA/RUBBER

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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