SVENSKA

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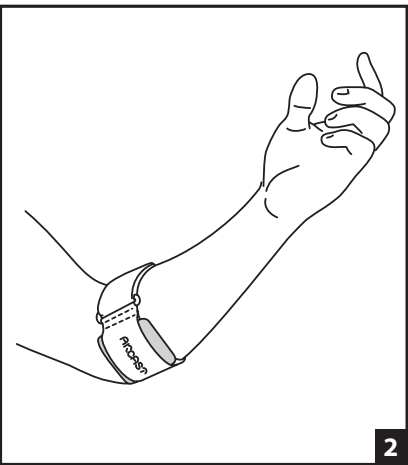
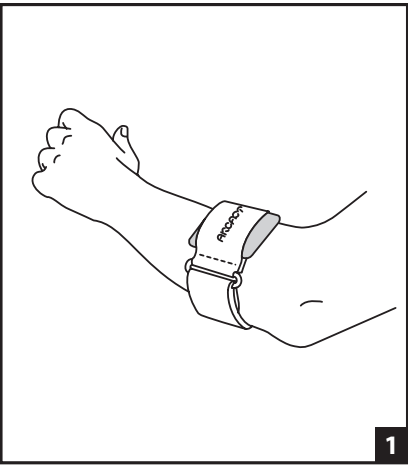
1430 Decision Street
Vista, CA 92081-8553 • USA

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS: For tennis elbow

PERFORMANCE CHARACTERISTICS:

- Soft-good / semi-rigid designed to restrict motion through elastic or semi-rigid construction.

- Providing mild compression of limb or body segment through elastic construction.

CONTRAINDICATIONS: N/A

WARNINGS AND PRECAUTIONS:

- If you have skin disorders / injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Excessive tightening of Pneumatic Armband may compromise circulation. If tingling or numbness is experienced, loosen strap.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

- Wrap armband around arm (Aircast logo faces out). Loosely secure strap.
- Adjust armband so aircell is centered above or below forearm.
- Readjust strap snugly.

CLEANING INSTRUCTIONS:

Hand wash in lukewarm water with mild soap. Air dry.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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