
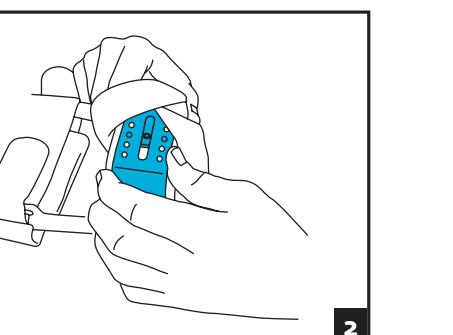
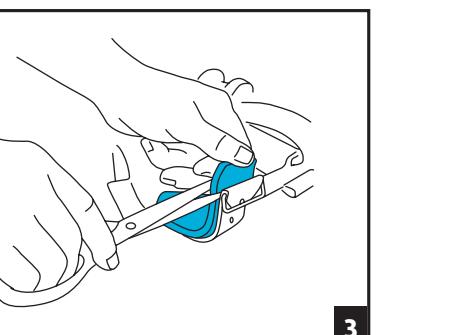
1


2

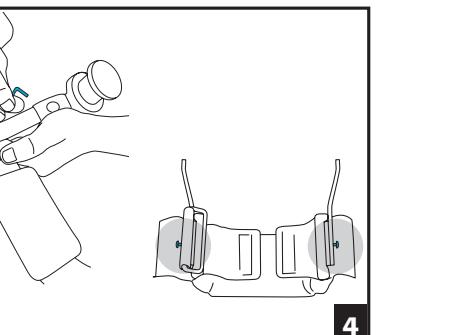
3

4

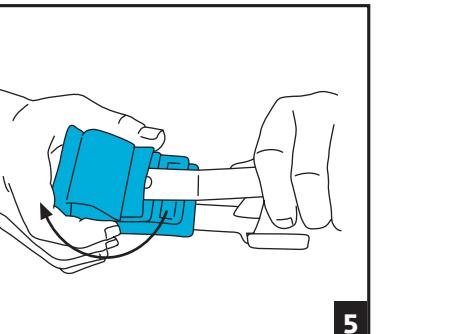5

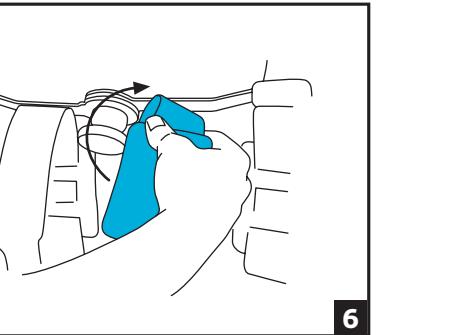6

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

FOR HEALTHCARE PROFESSIONAL USE ONLY

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS: For adjustment of the Mayo Clinic Elbow Brace o5EL and o5ER.

PERFORMANCE CHARACTERISTICS:

- Product soft/semi-rigid designed to restrict motion through elastic or semi-rigid construction. • Providing immobilization or controlled movement of the limb or body segment.

APPLICATION INFORMATION:

Adjust position of control knob: (The Mayo Clinic Elbow Brace is preset for application on the right arm or the left arm, the control knob on the medial side. However, the lateral side position may be adjusted to be on the lateral side depending on the patient's preference.)

1. Open the cloth cover on the medial and lateral sides of each end bracket.
2. Using an allen wrench (included with brace), loosen the two screws on the medial and lateral uprights of each end bracket.
3. Rotate each end bracket 180°.
4. Replace the cloth cover on each bracket making sure that the fabric completely covers the metal surfaces.
5. Completely open the forearm and upper arm straps and slide each strap out of the slot on the bracket so that it hangs freely. Be sure to open both the top portion of the straps as well as the bottom (the straps are two-sided).
6. One at a time firmly hold each strap near the rivet and rotate it 180°. Reinsert each strap into the slot.
7. **Adjust Brace Size:** (The Mayo Clinic Elbow Brace is preset to the extra large size, suitable for most patients. However, the brace may be adjusted to accommodate sizes ranging from extra large to extra small)
1. Open straps and cloth cover on each end bracket until the bracket screw is exposed.
2. Using an allen wrench (included with the brace), loosen the screw on each bracket.
3. Carefully lift the top portion of each bracket and reposition the silver pins into the desired holes marked by the numbers 1-5. The pins must be inserted into the same numbered holes on both brackets to ensure that the brace will work properly.
4. Retighten screws on each bracket.
5. If necessary, trim the gray foam padding so that it does not extend beyond the bracket.
6. Replace the cloth cover on each bracket making sure that the fabric completely covers the metal surfaces.

WARNINGS AND PRECAUTIONS:

- If you have skin disorders / injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional. • Do not use over open wounds. • If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or packaging has been opened.
- **NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.
- **WARNING:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.
- **RX ONLY**
- **FOR SINGLE PATIENT USE ONLY**

NOT MADE WITH NATURAL RUBBER LATEX.

U.S. PATENT #7517329

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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