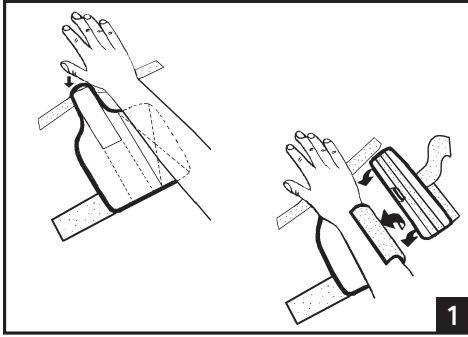
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for the user.

INTENDED USE/INDICATIONS: The Forearm Rotational Brace is used for pronation control with lateral collateral ligament injuries/repairs and pronation/contracture. Supination control is ideal for medial collateral ligament injuries/repairs and supination contracture.

PERFORMANCE CHARACTERISTICS:

- Self-poor semi-rigid designed to restrict motion through elastic or semi-rigid construction.
- Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS: N/A

WARNINGS AND PRECAUTIONS:

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

- 1. Apply Forearm Rotational Brace to hand and wrist.** The Forearm Rotational Brace should be secured to the hand and wrist before the Clinic Elbow Brace is applied.
 - Open straps completely and separate bottom (volar) wrist support from top (dorsal) wrist support. (Place the pronation/supination strap and D-ring strap on the side to be used later)
 - Lay bottom wrist support flat and place arm, palm side down, on top (Fig. 1).
 - Wrap side of bottom wrist support around wrist. Lay top wrist support on top of wrist and secure to bottom support (Fig. 2).
 - Wrap straps around wrist; first short strap then the long strap. Insert end of long strap into the ring and secure (Fig. 3).
 - Wrap finger straps around fingers and secure to top wrist support (Fig. 4).
 - Readjust all straps for a snug but comfortable fit. (Wrist shown in pronation)

2. Prepare Clinic Elbow Brace.

- Separate the pronation/supination strap from the D-ring strap (included with Forearm Rotational Brace).
- Attach the D-ring strap to the Clinic Elbow Brace bar.

For pronation, apply strap to medial bar of Elbow Brace.

For supination, apply strap to lateral bar of Elbow Brace.

Wrap the strap around bar in between the forearm strap and wrist bracket. Insert the free end of the strap into the loop, and pull through until secure (Fig. 5).

3. Apply Clinic Elbow Brace to arm

- Apply the Clinic Elbow Brace (see instruction sheet included with Elbow Brace). The Elbow Brace will overlap the Forearm Rotational Brace.
- Lock the Elbow Brace at 90°.
- **4. Attach Forearm Rotational Brace to Clinic Elbow Brace Pronation application**
 - Apply pronation/supination strap to top wrist support with free end toward thumb (Fig. 6).
 - Wrap strap around hand and wrist at a 45° angle (Fig. 7).
 - Insert free end of strap into D-ring on medial side of Elbow Brace (Fig. 7).
 - Pull strap through D-ring and firmly secure.
 - Tighten strap as tolerated to increase pronation.

Supination application

- Apply pronation/supination strap to top wrist support with free end toward little finger (Fig. 8).
- Wrap strap around hand and wrist at a 45° angle (Fig. 9).
- Insert free end of strap into D-ring on lateral side of Elbow Brace (Fig. 9).
- Pull strap through D-ring and firmly secure.
- Tighten strap as tolerated to increase supination.
- Do not use this device if it was damaged and/or packaging has been opened.

NOTE: When using the Forearm Rotational Brace for pronation/supination control, the strap may be too long for some patients. If necessary, adjust the placement of the strap on the top wrist support and cut off the excess strap.

CLEANING INSTRUCTIONS:

The Forearm Rotational Brace may be hand-washed in cold water with mild detergent and air-dried. Do not put in dryer or other heat source to dry.

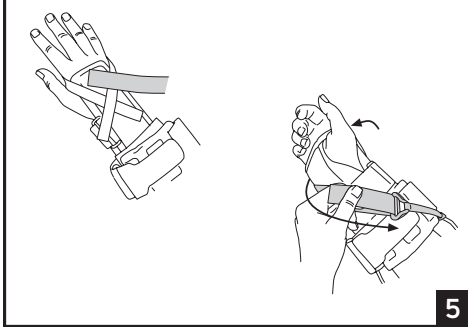
WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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