
LR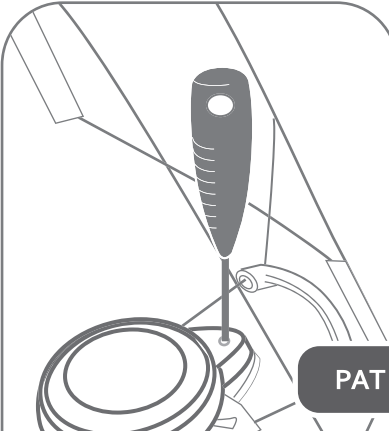

PATIENT INSTRUCTIONS

BOA® LOCKING RING

GIVE TO PATIENT

BOA® LOCKING RING
 ANILLO DE BLOQUEO BOA®
 BOA®-SICHERUNGSRING
 ANELLO DI BLOCCO BOA®
 ANNEAU DE VERROUILLAGE BOA®
 BOA® SLUITRING
 POISTNÝ KRÚŽOK BOA®
 BOA®-LÄSERINGEN
 BOA® LÄSRING
 BOA®-LUKITUSRENGAS
 POJISTNÝ KROUŽEK BOA®
 ANEL DE BLOQUEIO BOA®
 BOA®-LÄSERINGEN
 BOA® 扣環用于防止
 BOA® 固定リングは

LR_10139 REV F 2021-03-03

ENGLISH

Before using the device, please read the entire instruction manual. Correct application and care is vital to the proper functioning of the device.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE

The Boa® Locking Ring is intended to prevent the Boa® Reel on the Exos® Braces from being lifted to the unlocked position, thus inhibiting the use of the Boa® Reel.

⚠ WARNINGS AND PRECAUTIONS

- If the brace is applied with a Boa® Locking Ring, the patient must not get brace wet.

SECURING THE LOCKING RING

- Ensure Boa® Reel is in the down and locked position before applying Locking Ring.
- Insert hook of Locking Ring cap into the slot. (Figure 1 & 2)
- Warning: If the Boa® Reel is in the up position, the Locking Ring will not engage properly.**
- Use the small Boa® screwdriver key to secure the Locking Ring by turning the key clockwise until Locking Ring fully seats onto the Boa® Reel base. (Figure 3)
- Caution: Do not over tighten the screw.**

REMOVING THE LOCKING RING

- Loosen the screw on the Locking Ring by turning the screw driver key counter-clockwise. (Figure 4)
- Lift off Locking Ring. (Figure 5)

To learn about the care of your state-of-the-art Exos® Brace...

Visit the Patient section of our website at: <http://exosmedical.com/patient/LR> or Scan the QR image with your smartphone

LIMITED PRODUCT WARRANTY

DJO, LLC will repair and replace all or part of the unit and its accessories for material or workmanship defects for a period of 8 weeks from the date of sale.

To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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