## ITALIANO

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DJO, LLC  
1430 Decision Street  
Vista, CA 92081-8553 • USA

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE/INDICATIONS:** Medial epicondylitis (golfer's elbow) • Lateral epicondylitis (tennis elbow)

**CONTRAINDICATIONS:** None

### WARNINGS AND PRECAUTIONS:

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by products.
- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

### APPLICATION INFORMATION:

1. Grasp sleeve at the larger opening. Pull the sleeve onto the elbow so that the strap is at least two fingers width below the crease of the elbow.
2. Feed the strap through the D-ring and secure. Strap should be snug but not constrict.

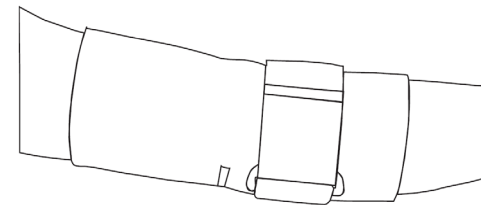
Figure 1

### CLEANING INSTRUCTIONS:

- Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

### FOR SINGLE PATIENT USE ONLY.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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