## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE/INDICATIONS:** For immobilization of the shoulder after surgery, injury or cast removal.

**CONTRAINDICATIONS:** None

### WARNINGS AND PRECAUTIONS:

- This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use.
- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

 **Caution:** This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.

### APPLICATION INFORMATION:

1. Place injured arm into the sling.
2. **Universal** - Bring straps over the shoulders crossing in the back. Feed the straps through the loop under the sling and tie with a bow. **Deluxe** - Take the straps over the shoulders and cross them in the back. Bring the straps under the sling feeding them through the buckles and secure. (Buckle tabs may be repositioned if desired) **Standard** - Wrap strap around neck and secure to the sling.
3. Attach the swathe to the front of the sling with the brushed material facing out. Wrap around chest and injured arm and secure.

**CLEANING INSTRUCTIONS:** Hand wash in warm water with mild detergent. Rinse thoroughly and air dry.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**FOR SINGLE PATIENT USE ONLY.**

 **NOT MADE WITH NATURAL RUBBER LATEX.**

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2021 DJO, LLC