## ITALIANO

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DJO, LLC  
1430 Decision Street  
Vista, CA 92081-8553 • USA

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE/INDICATIONS:** For mild compression and support of the elbow.

**CONTRAINDICATIONS:** None

### WARNINGS AND PRECAUTIONS:

- If you have skin disorders/ injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

### APPLICATION INFORMATION:

1. Pull sleeve onto arm with the larger opening first. Position the sleeve so it covers the affected area as shown. (Fig. 1)

Figure 1

### CLEANING INSTRUCTIONS:

- Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY. **NOTE:** If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

### FOR SINGLE PATIENT USE ONLY.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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