ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, or the patient's caregiver. The user or caregiver should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

INDICATIONS: To assist cooperative patients that require minimum support in a wheelchair or while walking. Not intended for bed use.

PERFORMANCE CHARACTERISTICS: Soft-good/semi-rigid designed for ambulation and/or transfer assistance.

CONTRAINDICATIONS: Never apply a Gait Belt if there is any compromise of safety or indication of inappropriateness to the patient or caregiver. If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.

WARNINGS AND CAUTIONS:

- If you experience any pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your medical professional immediately.

- Do not use over open wounds.

- Do not use this device if it was damaged and/or package has been opened.

- Do not use on patients with post-operative incisions, ostomy, colostomy, G-tubes, hernias, or severe cardio obstructive pulmonary disease.

- Do not use product as a restraint.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

A) Secure patient in wheelchair by wrapping belt around upper-abdominal or chest area and secure belt behind wheelchair back.

Item No.	Size
79-91000	Universal

USE AND CARE:

- Check restraint condition before and during use. Do not use if torn, worn, frayed or damaged in any way.
- Hand or machine wash with mild detergent in cold water. Air dry. Do not bleach.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

For single patient use only.

 **Not made with natural rubber latex.**

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2021 DJO, LLC