**ENGLISH**

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

**INTENDED USER PROFILE:**

THE USER SHOULD BE ABLE TO:

- READ AND UNDERSTAND THE DIRECTIONS, WARNINGS AND CAUTIONS.

**INTENDED USE/INDICATIONS:** Restricts cervical spine flexion, extension and rotation.**CONTRAINdications:** None

**WARNINGS AND CAUTIONS:** If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

**SIZING INFORMATION:**

1. For Adult sizing, use the SIZING GUIDE to determine the appropriate collar size by measuring the vertical distance from the tip of the chin to the top of the shoulder. For Pediatric sizing, reference the back of the SIZING GUIDE. (Figure 1)

**APPLICATION INFORMATION:**

2. SUPINE APPLICATION: To fit the patient when laying down.
  - A. Introduzca la PIEZA TRASERA por debajo del cuello del paciente y céntrala. Asegúrese de que el acolchado de espuma negra/gris sobresalga por el borde del plástico. (Figura A)
  - B. Flare the sides of the FRONT PIECE out, slide it up the chest and scoop it up under the chin. Sides of the FRONT PIECE should be oriented up, off the trapezius and toward the ears. (Figure B)
  - C. While holding the FRONT PIECE securely, curl the ends snugly against the patient's neck. Apply the Velcro strap firmly. Color the correa de VELCRO™ en la misma forma. Apriete las correas una después de la otra y de forma que la longitud sea la misma a ambos lados. Asegúrese de que la PIEZA DELANTERA del collar quede dentro de la PIEZA TRASERA. (Figure C)
  - D. To remove the collar, make sure the patient's head is in a neutral position and unfasten the hook-and-loop strap. Slowly slide the collar from behind the neck and remove. (Figure D)

**NOTA:** Collar application must be secure to ensure proper fit, maintaining prescribed alignment, and to prevent the chin from slipping inside. If the patient can slip his/her chin inside the collar, it is a clear indication that it is not snug enough.

3. UPRIGHT APPLICATION: To fit the collar to a sitting or standing patient.

- A. With the patient in neutral head alignment, slide the FRONT PIECE of the collar up the patient's neck and properly position the chin support. Sides of the collar FRONT PIECE should be oriented up, off the trapezius and toward the ears. (Figure D)
- B. Hold the FRONT PIECE firmly with one hand. Place BACK PIECE under crevice of neck until panel is centered and attach both sides to FRONT PIECE. (Figure E)
- C. Apply the Velcro strap and secure the opposite side of the collar in the same fashion. Tighten straps alternately, one at a time, to an equal length on both sides. Make sure the FRONT PIECE of collar is inside the BACK PIECE. (Figure F)

- D. To remove the collar, make sure the patient's head is in a neutral position and unfasten the hook-and-loop strap. Slowly slide the collar from behind the neck and remove. (Figure G)

**OTHER APPLICATION INFORMATION:**

**Angulation Buttons:** The ANGULATION BUTTONS may be used to fine-tune the fit of the collar for anatomical conformity and pressure distribution. It is easier to adjust the ANGULATION BUTTONS if you first remove the collar. (Figure G)

**Chin Support Positioning:** Slide up to help relieve chin tip pressure/discomfort. Slide down to help relieve jaw pain/discomfort.

**Occiput Support Positioning Guidelines:** Slide up to help enhance occipital pressure/discomfort. Slide down to help enhance extension control. For a predominantly supine patient, keep OCCIPITAL SUPPORT BUTTONS set to top position to help reduce pressure.

**CLEANING INSTRUCTIONS:** Separate the brace's removable pads before cleaning.

Hand wash in water (86°F/30°C) using soap. Air dry.

**MATERIAL CONTENTS: RIGID SHELL:** Low Linear Density Polyethylene; **SOFTCOATS:** UBL, Polyester, Polyurethane, Interlock.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

**Rx ONLY.**  
**INTENDED FOR SINGLE PATIENT USE.**  
**NOT MADE WITH NATURAL RUBBER LATEX.**

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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