ITALIANO

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13-3051-0-00000 REV D

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS: The Procure Humeral Fracture Brace is designed to provide support to the humerus following stable humeral diaphyseal fractures, prophylactic application after removal of internal fixation. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS: None

WARNINGS AND CAUTIONS: • If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately. • Excessive tightening of the brace straps may compromise circulation. If tingling or numbness is experienced, loosen straps. • Do not use over open wounds. • If pain, swelling, changes in sensation or other unusual reactions occur while using this product, consult your medical professional immediately. • Do not use this device if it is damaged and/or the packaging has been opened. **NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS:

A Sit in a chair. Place stockinette over upper arm. **Fig. 1**

B Loosen the two hook and loop straps on the brace. Slide the brace over the arm so it rests comfortably on the biceps. (The shorter side of the brace should be placed under the armpit.) **Fig. 2**


C Tighten straps beginning with the strap closest to the elbow then the upper strap. The straps should be cinched to produce a snug, comfortable fit. **Fig. 3**

D Straps should be adjusted throughout the day to ensure the snug fit is properly maintained.

Fig. 1

Fig. 2

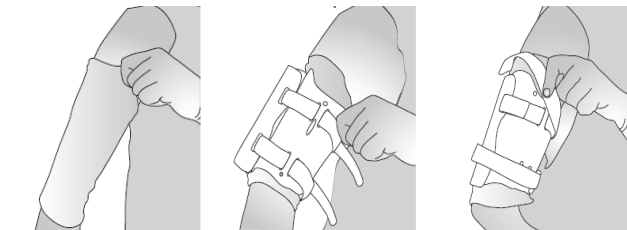Fig. 3

Fitting Chart

Size	Biceps Circumference	Brace Length	
		Medial	Lateral
Small	8" - 11"	5 ½"	8 ½"
Medium	11" - 14"	5 ½"	8 ¾"
Large	14" - 15"	6"	9 ¾"
XLarge	15" - 16"	14"	10 ¾"

USE AND CARE

• Remove the brace while seated. • The brace may be washed using a mild detergent in warm water. Rinse thoroughly and towel dry. • The stockinette may be washed in warm water with a mild detergent. After thorough rinsing, allow the stockinette to air dry. • Be sure you bathe with the brace on. **ONCE YOU HAVE DRIED OFF AND ARE SEATED THEN:** remove the brace, • dry it off, • apply a dry stockinette, • reapply the brace

NOT MADE WITH NATURAL RUBBER LATEX

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2020 DJO, LLC