## ITALIANO

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DONJOY®

# TRU-PULL™ ADVANCED SYSTEM PATELOFEMORAL SLEEVE

MANGA PATELOFEMORALE TRU-PULL™ ADVANCED SYSTEM  
TRU-PULL™ ADVANCED SYSTEM- PATELOFEMORAL ORTHOSE  
FASCIA PATELOFEMORALE TRU-PULL™ ADVANCED SYSTEM  
GENOUILLÈRE FÉMORO-PATELLAIRE TRU-PULL™ ADVANCED SYSTEM

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13-3717-0-00000 REV B - 2020-12-16

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**INTENDED USE/INDICATIONS:** The DonJoy Tru-Pull Advanced System Patellofemoral Sleeve is designed to support the knee and may be suitable for use following knee injuries such as dislocations. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

**CONTRAINDICATIONS:** N/A

### APPLICATION INFORMATION:




**A)** Apply sleeve on leg. Situate patella opening over knee cap. (Figure 1)

There are two buttresses to choose from, select the most comfortable buttress. Position the buttress properly on the medial or lateral border of the patella to address indication.

**B)** Take the bottom strap and wrap it around the calf and affix the hook to the sleeve. (Figures 2 & 3)

**C)** Take the top strap and wrap it around the thigh and affix the hook to the sleeve. (Figures 2 & 3)

**D)** Adjust the tension on the patella by pulling the straps tighter around the thigh and calf.

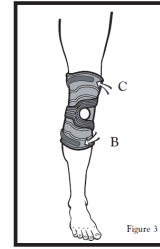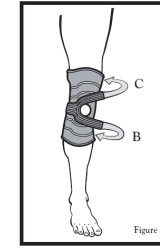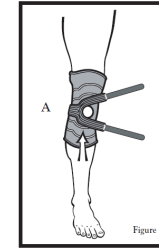### USE AND CARE:

Hand wash in warm water with mild soap, rinse thoroughly. AIR DRY. **Note:** If not rinsed thoroughly, residual soap can cause irritation or deteriorate material. Brace should not be used if it fails to provide relief or if there is excessive wear. Product for multiple uses on a single patient. Cleaning of brace recommended and advocated. Not for use in water sports. Product to be checked for wear before each use.

**WARNINGS AND PRECAUTIONS:** This product is to be used under the supervision of a medical professional.

- Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by products.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.
- If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

FOR SINGLE PATIENT USE ONLY.

NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2020 DJO, LLC