ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INDICATIONS: The Procure Xceltrax Air Walker is designed to provide support and immobilization to the lower leg, ankle and foot. It may be suitable for the treatment of stress fractures of the lower limb, stable ankle and/or foot fractures, Achilles tendon repair and acute lateral ankle sprains. Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS: Do not use if you are allergic to any of the materials contained within this product. This device is contraindicated for unstable fractures of the lower leg, ankle and foot.

WARNINGS AND PRECAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

Do not overinflate aircells. Improper aircell inflation may cause significant irritation in patients with diabetic neuropathy. Reduce aircell pressure with any visual skin changes or reported discomfort. Do not use this device on patients incapable of communicating physical discomfort.

Like all lower extremity immobilizers, such as casts or braces, patients without sensation (ie: post-op anesthesia, neuropathies, etc.) should be monitored frequently for skin damage, skin irritation or wound management.

Use caution when walking on slippery or wet surfaces to avoid injury.

At high altitudes the aircells will expand beyond optimal level. Adjust aircell support by tightening or loosening straps.

Always inspect before use. Check for broken stitches or parts; torn, cut or frayed material; or buckles, or hook and loop fasteners that do not hold securely. Do not use damaged product. Discontinue use when torn, worn, frayed in any way.

Never alter or repair this product or use the device other than intended. This product is not intended to be used with any other product except DJO approved accessories.

Use a sneaker or street shoe with a ShoeLift™ or equivalent on uninjured foot while ambulating to compensate for leg length changes while wearing a fracture walking brace.

APPLICATION INFORMATION:

- PNEUMATIC Walking Brace**
1. Unfasten straps and remove liner from boot. (Fig.1)
 2. Place foot in liner with the heel fitting snugly into the back portion of the liner. Fasten and wrap foot flaps then the leg portion of the liner. Liner should be snug from top to bottom but should not constrict. (Fig.2)
 3. With the colored "pull up" tabs intact, gently spread the side struts and place the foot onto the foot plate heel first, then the forefoot. The ankle should be centered at the struts with the foot and ankle at 90°. (Fig.3)
 4. Once in the correct alignment, remove the colored "pull up" tabs and gently push the struts towards the liner to ensure the hook and loop on the walking brace is making contact with the liner. (Fig.4)
 5. Fasten the hook and loop straps beginning at the toe then moving up the limb. Do not over tighten. As swelling patterns change it may be necessary to adjust the straps. (Fig.5)
 6. To inflate air liner, press and release center air bulb until liner provides a comfortable cushion and support. Do NOT over inflate. To deflate or adjust the level of support, press and hold the deflate button and deflate until pressure is comfortable. (Fig.6)
 7. To remove, deflate air cells and unfasten all straps without removing them completely from the assembled walker. Unfasten the soft good liner and gently bring the foot and ankle forward and out of the walker. To reapply, place the foot and ankle into the liner, reattach the Velcro closures, and inflate air liner for comfortable fit. (Fig.7)

• This device is to be used under the supervision of a healthcare professional. The determination of when to apply the brace and the frequency and duration of use should be strictly at the discretion of the treating physician. Always consult with your physician before making changes to the brace.

• Application of this device is recommended only when the fracture is demonstrably stable and there area acceptable limits of angular and rotational deformity.

• Do not use over open wounds.

• Do not over tighten straps. This may result in reduction of blood flow or sensation.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

NOTE: For added comfort, the extra rectangular shaped pads may be used for extra padding inside the liner, under the straps, between the walking brace shell and the liner, etc.

CLEANING INSTRUCTIONS: Hand wash liners using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOT MADE WITH NATURAL RUBBER LATEX

**SINGLE PATIENT USE
RX only.**

COMPONENT	MATERIAL
Frame	Injection molded glass filled Nylon; Steel
Liner	Foam laminate; Polyester
Outsole	Injection molded TPR
Insole	EVA foam

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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