APPLICATION INSTRUCTIONS: AMBULATORY

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INTENDED USE/INDICATIONS: The Trend Collar is designed to stabilize and relieve the cervical spine and may be suitable for the symptomatic relief of pain and stiffness in the cervical spine area following patient trauma or injury. It may help to minimize neck pain and discomfort. For further details, please consult your prescribing physician.

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AMBULATORY (UPRIGHT) PATIENT APPLICATION: (Note: Follow same process for Hook & Loop Version)

Step 1: Position patient into neutral head alignment
Position the patient into neutral head alignment. Ensure tip of nose is aligned with bottom of earlobe and parallel to ground.

Step 2: Prepare the Collar
Secure one side of the back panel to the front panel by attaching one of the clips into the corresponding clear plastic slots on the front panel. (Note: attach hook strap tab to loop strap tab)

Step 3: Apply Collar to Patient
3a. With patient in neutral head alignment, slide the front panel of the collar under the patient's chin and properly position the chin support. 3b. Fasten the strap clip into the clear plastic slot until you hear a click. (Note: attach hook strap tab to loop strap tab)

Step 4: Adjust the Height
Unlock the front dial by pulling it outward. Stabilize the distal portion of the anterior panel to the sternum. To adjust the chin height to a lower position, turn the dial clockwise. To adjust it higher turn the dial counterclockwise. Once the foam of the chin piece gently compresses against the patient's chin push the dial inward to lock it in place.

Step 5: Adjust the Circumference
To adjust circumference, release both the left and right hook & loop. now pull both left and right straps uniformly until the desired tension is achieved.

Step 6: Adjust as needed
If necessary, make minor adjustments to the straps and the front and back panel supports to improve comfort, fit and immobilization.

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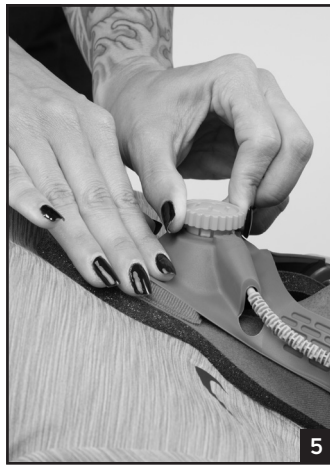
TRAUMA TX COLLAR TRAUMA COLLAR MULTI-POST COLLAR

COLLARÍN TRAUMA TX COLLARÍN TRAUMA MULTIPOSTE
COLLIER TRAUMA TX COLLIER TRAUMA COLLIER MULTI-POST

APPLICATION INSTRUCTIONS: SUPINE

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. User should be able to:

- Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS: The Trend is designed to stabilize and relieve the cervical spine and may be suitable for the symptomatic relief of pain and stiffness in the cervical spine area following patient trauma or injury. It may help to minimize neck pain and discomfort. For further details, please consult your prescribing physician.

CONTRAINDICATIONS: Do not use if you are allergic to any of the materials contained within this product.

WARNINGS AND CAUTIONS:

- This product is to be prescribed and fitted by a physician.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.
- The collar should be snug but not tight enough to cause excessive pressure, pain or difficulty breathing or swallowing.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact a medical professional immediately.
- There could be difficulties with laryngoscopy when collars are used; hence, the device may have to be removed.

SUPINE PATIENT APPLICATION:

(Note: Follow same process for Hook & Loop Version)

Step 1: While patient is immobilized in C-Spine
While maintaining C-spine immobilization.

Step 2: Position Back Panel behind Patients Neck
Press the back panel down onto the bed and slide it behind the patients neck. The straps should be between the earlobes and top of shoulders.

Step 3: Position the front panel
With patient in neutral head alignment, position the front panel under patient's chin and ensure the chin rests centrally on the chin support of the collar.

Step 4: Attach front & back panel
To attach the front section of the collar to the back panel, tuck the side edges in and secure using the quick clip closure system. Do this by locating the clips on both sides and inserting until you hear a click. (Note: attach hook strap tab to loop strap tab)

Step 5: Adjusting the Height
Unlock the front dial by pulling it outward. Stabilize the distal portion of anterior panel to sternum. Then to adjust the chin height to a lower position, turn the dial clockwise. To adjust it higher turn the dial counterclockwise. Once the foam of the chin piece gently compresses against the patient's chin push the dial inward to lock it in place.

Step 6: Adjust the circumference
To adjust circumference, first stabilize the distal portion of the anterior panel to the sternum. release both the left and right hook & loop and pull them uniformly until the desired tension is achieved. Both letters in the sizing window should match.

Step 7: Adjust as needed
If necessary, make minor adjustments to the straps and the front and back panel supports to improve comfort, fit and immobilization.

CLEANING & STORAGE INSTRUCTIONS:

 Hand wash in warm water (86°F/30° C) using mild soap and rinse thoroughly. AIR DRY.

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate materials.

MATERIAL CONTENTS: Nylon, Polyester, Propylene Ethylene Copolymer, Thermoplastic Polyester Elastomer, Polyurethane, Polyethylene.

WARRANTY: 1 year from time of purchase.

 **NOT MADE WITH NATURAL RUBBER LATEX.**

For more information on the Trend Collar, please visit www.djoglobal.com.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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