Figure 1

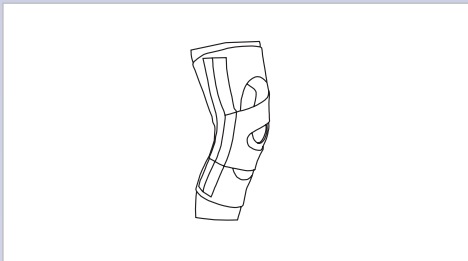
Figure 2

## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE

The Lateral J Patella Support is designed to provide support to the knee. It may be suitable for those suffering from instabilities and knee pain. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

### INTENDED USE/INDICATIONS

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### PERFORMANCE CHARACTERISTICS

Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

### CONTRAINDICATIONS

- Phlebitis patients should not wear neoprene products.

### WARNINGS AND PRECAUTIONS

- Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by-products.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.
- If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INSTRUCTIONS

1) Grasp edges of the sleeve at the larger opening. Step through sleeve, bringing it on to the knee so that the opening at the front surrounds the kneecap. (Fig. 1)

2) Wrap and attach the bottom then top neoprene strap to the inside of the knee. (Fig. 2)

### CARE

Hand wash in warm water using mild soap, rinse thoroughly, AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

 FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.

**WARRANTY** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**NOTICE:** WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2020 DJO, LLC
MDSS GmbH