## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use

**INTENDED USE / INDICATIONS:** The Exos® Wrist Undersleeve is designed to be used under the Exos® wrist and short arm fracture braces to provide comfort and protection to the patient's skin from shear and moisture.

**PERFORMANCE CHARACTERISTICS:** Providing mild protection to the limb or body segment.

**CONTRAINDICATIONS:** None.

**WARNINGS AND CAUTIONS:** This product is to be prescribed and fitted by a healthcare professional. The frequency and duration of use should be determined by your healthcare professional.

- Do not use device other than instructed in this IFU.
- Do not modify the device without a healthcare professional present.
- Do not use over open wounds.
- Do not use this device if it is damaged and/or the packaging has been opened.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, consult your medical professional immediately.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device

### CLEANING INSTRUCTIONS:

Hand wash only. Do not iron.

### Rx ONLY.

INTENDED FOR SINGLE PATIENT USE

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2021 DJO, LLC
MDSS GmbH