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DJO, LLC  
1430 Decision Street  
Vista, CA 92081-8553 • USA

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USER PROFILE:** The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE/INDICATIONS:** • Post-operative rotator cuff repairs • Post-operative Bankart procedures • Capsular shifts • Glenohumeral dislocation/subluxation • Soft tissue strains/repairs.

**This product was designed to complement the variety of medical treatments common to the above afflictions. The frequency and duration of use should be determined by your prescribing Healthcare professional.**

**PERFORMANCE CHARACTERISTICS:** Providing immobilization or controlled movement of the limb or body segment.

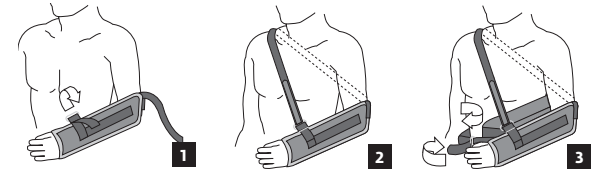
**WARNINGS AND PRECAUTIONS:**

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- This product is to be used under the supervision of a medical professional.
- This device is not intended for unsupervised public use.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

**APPLICATION INFORMATION:**

- Detach the shoulder strap and open front panel. Place the arm in the sling with the elbow as far back into the sling as possible. Secure the front panel strap through the lower D-ring (Fig. 1).
- Affix the snaps along the top of the sling to secure the arm in its cradle. Note: The thumb may be placed through the loop inside the sling for additional comfort.
- Bring the shoulder strap across the back and over the unaffected shoulder and fasten through the middle D-ring of the sling. The shoulder strap may be trimmed by removing the Y-tab at the end of the strap, cutting the strap and reapplying the Y-tab (Fig. 2).
- Place the pillow at the waistline with the large end facing forward. Attach the sling to the pillow, lining up the hook and loop strips so they adhere. Secure the waist strap through the D-ring on the front end of the pillow (Fig. 3). **NOTE:** The position of the pillow should be positioned for desired internal or external rotation by sliding the pillow forward or back along the waistline.

**USE AND CARE:**

- Sling and pillow cover may be washed in cold water with mild detergent. Air dry. Always consult with your physician or therapist before making changes to the brace.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

**FOR SINGLE PATIENT USE ONLY**

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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