ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INDICATIONS: The DonJoy Sports Knee Brace Cover is designed to provide protection to the brace and others from hitting the brace during contact sports. Providing mild protection of a limb or body segment for acute and prophylactic care.

APPLICATION INFORMATION:

A) First apply the Defiance or Intrepid brace as instructed in the application instructions that come with the brace.

B) Pull the sliding sleeve on over the brace while making sure the DonJoy logo is positioned on the front side of the leg and above the knee.

C) Minor adjustments may be required to insure the padding is oriented to cover the rigid portions of the brace.

CONTRAINDICATIONS: None

USE AND CARE:

Hand wash in cold water using mild soap. Rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause skin irritation and deteriorate the material.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

WARNINGS AND PRECAUTIONS:

- Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by products.
- If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

FOR SINGLE PATIENT USE ONLY.

NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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