**DONJOY®**

# ULTRASLINGTM II/III DE-ROTATIONAL STRAP

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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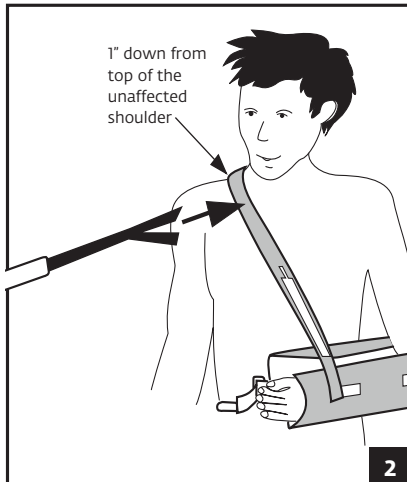
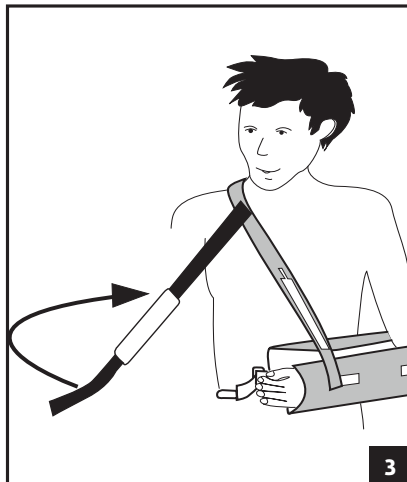
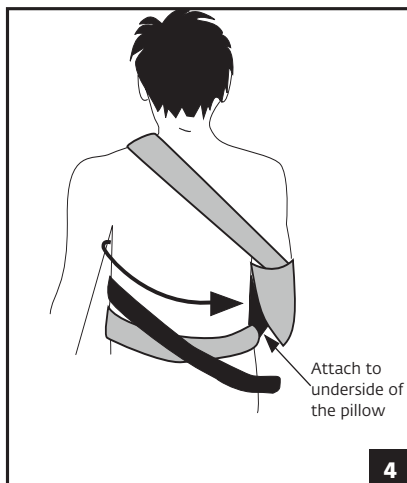
4

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USER PROFILE:** The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE/INDICATIONS:** Postoperative rotator cuff repairs, Postoperative Bankart procedures, Capsular shifts, Glenohumeral, dislocation/subluxation, Soft tissue strains/repairs.

**PERFORMANCE CHARACTERISTICS:** Providing immobilization or controlled movement of the limb or body segment.

**CONTRAINDICATIONS:** None

**WARNINGS AND PRECAUTIONS:**

• If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional. • Do not use over open wounds.

If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately. • Do not use this device if it was damaged and/or packaging has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

**CLEANING INSTRUCTIONS:** Hand wash in cold water using mild soap, rinse thoroughly, AIR DRY. **NOTE:** If not rinsed thoroughly, residual soap may cause irritation and deteriorate material. The waist and shoulder strap may be trimmed by removing the Y-tab Ultrasing II/III only).

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©2021 DJO, LLC