**ENGLISH**

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USER PROFILE:** The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE/INDICATIONS:** Provides cushioned support for patient limb. Traps fluids from patient, prevents pooling on O.R. surface area and expedites post-operative clean-up. May be used post-operatively as an insulated positioner for cryotherapy applications.

**PERFORMANCE CHARACTERISTICS:** Soft-good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.

**CONTRAINDICATIONS:** NONE

**WARNINGS AND PRECAUTIONS:**

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Discontinue use when torn, worn, frayed or damaged in any way.
- Always inspect product prior to application to ensure all parts are in proper working order.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

**APPLICATION INFORMATION:**

- 1) Open straps on protector with white interface lining side up. Position patient's arm so the Elbow rests on the center of the protector.
- 2) Bring straps over the patient's arm and attach straps to hook tabs. Protector is now secure.
- 3) Place arm in desired position.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**FOR SINGLE PATIENT USE ONLY**

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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