



# PROCARE® XTEND 174 PEDIATRIC

## Clinician Instructions



### APPLICATION & CARE INSTRUCTIONS:

BEFORE USING THIS MEDICAL DEVICE, PLEASE READ THE **INSTRUCTIONS FOR USE** COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE. THIS PRODUCT IS INTENDED TO BE USED UNDER THE SUPERVISION OF A HEALTHCARE PROFESSIONAL.

### PEDIATRIC SIZING GUIDE

- 0** AGE: 0 - 6 mos
- 1** AGE: 6mos - 2 years    Height: 1.125", Neck Circum: ≤9"
- 2** AGE: 2 - 6 years        Height: 1.25", Neck Circum: 8" - 11"
- 3** AGE: 6 - 12 years       Height: 1.125", Neck Circum: 9" - 13"

### REMOVAL:

- 1** Detach hook-and-loop strap from **one side** of the collar and carefully remove ProCare® XTEND 174 Pediatric.

### RE-APPLICATION FROM UPRIGHT POSITION:

- 2** While sitting or standing with head in neutral position, wrap ProCare® XTEND 174 Pediatric cervical collar around neck. (Figure 2)
- 3** Apply hook-and-loop and then tighten straps alternately, one at a time, to an equal length on both sides until product is centered comfortably. Make sure the FRONT PIECE of collar is inside the BACK PIECE. (Figure 3)

**NOTE:** Collar application must be secure to ensure proper fit, maintaining prescribed alignment, and to prevent the chin from slipping inside. If the chin can slip inside the collar, it is a clear indication that it is not snug enough.

### CLEANING INSTRUCTIONS:

Separate the brace's removable pads before cleaning. Hand wash in water (86°F/30°C) using soap. Air dry.

### MATERIAL CONTENTS:

RIGID SHELL: Low Linear Density Polyethylene;  
SOFTGOODS: UBL, Polyester, Polyurethane Foam, Interlock.

### WARRANTY:



DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

Rx ONLY.

INTENDED FOR SINGLE PATIENT USE.

NOT MADE WITH NATURAL RUBBER LATEX.

**WARNINGS AND PRECAUTIONS:** If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

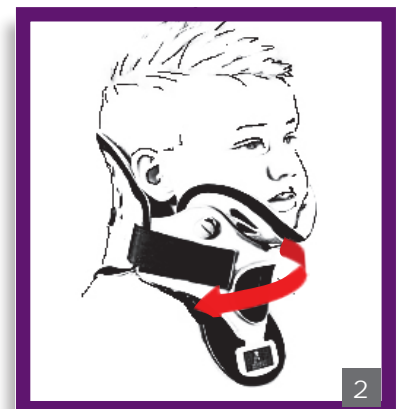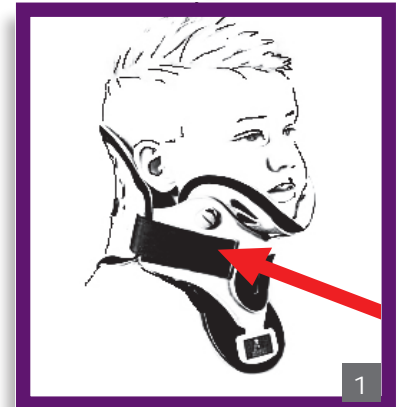

DJO, LLC | A DJO Global Company

T 800.336.6569    D 760.727.1280    F 800.936.6569  
2900 Lake Vista Drive | Lewisville, TX 75067 | U.S.A.

[DJOGlobal.com](http://DJOGlobal.com)

Individual results may vary. Neither DJO, LLC nor any of its subsidiaries dispense medical advice. The contents of this poster do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.

COPYRIGHT©2019 DJO, LLC

MKTIS00-9244 Rev A