ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INDICATIONS: Medial epicondylitis (golfer’s elbow) • Lateral epicondylitis (tennis elbow)

APPLICATION INFORMATION

- 1) Wrap support around the forearm at least two finger widths below crease of the elbow.
- 2) For supports with compression pads, place pad on affected side of the forearm. Support should fit snug, but not tight enough to impair circulation.
- 3) Loop end through D-ring and secure. For tennis elbow wrap, secure hook and loop closures. Support should fit snug, but not tight enough to impair circulation.

USE AND CARE

Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

WARNING: Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by products.

WARNING: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

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