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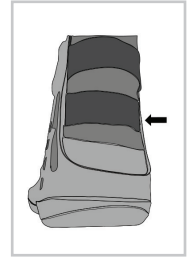
Figure 5

## ENGLISH

For first time application, loosen all straps and remove liner from boot.

1. Removable foam ankle pads are provided for patient comfort (Figure 1).
2. Place foot in liner and secure with contact closure. Make sure the heel fits snugly into the posterior portion of the liner. Secure the forefoot flap on the liner first followed by the lower leg portion. Make sure liner fits snug top to bottom (Figure 2).
3. Spread the uprights using both hands and step into boot, aligning uprights with midline of the ankle. Remove protective sheaths on uprights and secure liner. The uprights may be bent to accommodate any leg contour (Figure 3).
4. Secure all bootstraps, starting at the toes and working up the leg (Figure 4A).

**Note:** All straps are removeable and replaceable. The two straps at the ankle joint may be adapted to accommodate individual patient needs.

**Ankle Cross Strap:** If Cross Strapping is desired, feed Strap 3 through opposing forefoot d-ring. Cross Strap 2 over Strap 3, feed through d-ring on opposing upright and secure (Figure 4B).

5. The Achilles (Heel strap) should be snug but not too tight. It may also be removed (Figure 5).
6. For removal of walker, unfasten all straps and undo contact closure on liner.
7. For reapplication of walker, follow steps 2, and 4.

## WARRANTY:

DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**NOTICE:** WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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COPYRIGHT © DJO, LLC 2021

MKTIS00-11470 Rev A

**Individual results may vary.** Neither DJO LLC, nor any of its subsidiaries dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.

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T 800.336.6569 | D 760.727.1280 | F 800.936.6569 | 5919 Sea Otter Place, Suite 200 | Carlsbad, CA 92010-6750 | U.S.A.