

BEFORE APPLYING THE BRACE, PLEASE READ THESE INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO PROPER FUNCTIONALITY OF THE BRACE.

INTENDED USE:


This brace is designed to alleviate pain by supporting the body in guiding and stabilizing the knee joint.


INDICATIONS:


For patients with symptoms, physical findings or radiographic evidence of moderate to severe unicompartmental knee osteoarthritis, or post-operative meniscal tears only where unicompartmental unloading is indicated.


APPLICATION OF THE BRACE:


- 1.** Disconnect magnetic latches on Thigh and Calf



- 2.** While sitting on the edge of a chair, flex the leg at 45 degrees, position hinge at top of the knee cap and slightly in front of midline of the leg on the affected compartment


- 3.** With buckles in the open position, connect the magnetic latches on Thigh and Calf


- 4.** Bring Thigh and Calf buckles to closed position


- 5.** Adjust Thigh and Calf circumferential straps as necessary


- 6.** For Thigh tensioning, move leg back into flexed 45 degree position, rotate BOA® dials clockwise to tighten as necessary, note location on printed graduated scale. Repeat for calf



Next appointment:

Date: _____
Time: _____

Notes: _____



USE AND CARE:

- This product is intended for single patient use only.
- If during activity, re-tightening is necessary, re-tighten all straps following steps in the application instructions.
- The brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water and then air dry.
- Hand wash your liners, straps and condyle pads in coldwater with mild detergent. AIR DRY ONLY; DO NOT HEAT DRY LINERS, STRAPS OR PADS. Use wet wash cloth with mild detergent to hand wash uprights and shell. Rinse entire brace in cold water. AIR DRY.
- Lubricate hinges periodically with dry lubricant or Teflon™ spray.
 To lubricate internal hinge mechanism, place brace in flexion and spray into the back side of the hinge cover.
- This device is not intended to be used with other devices except those specifically listed as accessories to this device.
- CONTACT BRACE FITTER OR CLINICIAN for any brace modifications or if screws become loose.

Always consult with your physician or therapist before making changes to the brace.

Digital Resources:



Webpage



Datasheet



Patient Application
Video

Intended for use on a single patient.

Rx Only

LATEX FREE



DJO, LLC

T 800.336.6569 F 800.936.6569

5919 Sea Otter Place, Suite 200 | Carlsbad, CA 92010 | U.S.A.

DJOglobal.com