

enovis™

Parent Company of DJO, LLC

MANAFUSE®

QUICK START GUIDE

Read prior to use.



QUICK START GUIDE

READ PRIOR TO USE. FOLLOW STEPS 1-16 BELOW.

Prescription Only -
CAUTION: Federal Law restricts
this device to sale by or on the
order of a physician.



1. If not already connected, connect the transducer cable at the top of the ManaFuse device.



2. Grasp the strap with the cap facing upwards and thread the strap's end through the loop's bottom.



3. Press the sides of the cap to release and open.



4. Arrange the strap so the cap is directly aligned with the fracture, as directed by your healthcare provider.



5. Fasten the strap firmly using the hook-and-loop mechanism.



6. Take off the cap from the gel bottle, grasp the transducer by the cable under the head, and dispense one pump of gel onto the surface.



7. Place transducer so the gel faces down in the transducer holder.



8. Position the transducer making sure the gel is touching skin.



9. Position the cap's notch in line with the transducer cord and close securely. Put the cap back on the gel bottle.



10. Press the left Power button to power on the ManaFuse device.



11. Press the Ultrasound button to start 20 minute treatment.



12. After treatment, open cap and gently remove transducer.



13. Carefully remove all residual gel from skin with a cloth. Remove any excess gel from transducer with a soft cloth. Do not use any cleaning fluid.



14. Remove any excess gel from transducer holder with a soft cloth.



15. Remove any excess gel from transducer holder cap with a soft cloth.



16. To charge device, insert the power cord into the bottom of ManaFuse and plug other end into outlet.

Customer Care:
Ph: 800.263.6004
Email: CMFQualityReturns@enovis.com

Web: <https://enovis.com/manafuse>

enovis™

T 800.263.6004

DJO, LLC
5919 Sea Otter Place, Suite 200 | Carlsbad, CA 92010 | U.S.A.
enovis.com/regeneration

Copyright © 2025 DJO, LLC
MKT00-14158 Rev A



Individual results may vary. Neither DJO, LLC nor any of the Enovis companies dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.